


The Commuter reusable face mask




2 layers of high quality
quilter's cotton

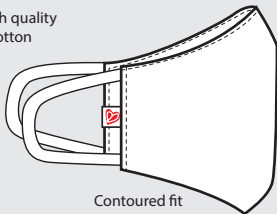

Contoured fit

Easy on, easy
off elastic
ear loops

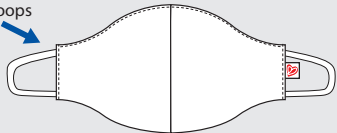


How to wash your mask:

Put it directly into the
washing machine.

Wash with other items using a hot
cycle with laundry detergent,
and lay flat to dry thoroughly.

Wash your hands after putting the
face covering into the laundry.

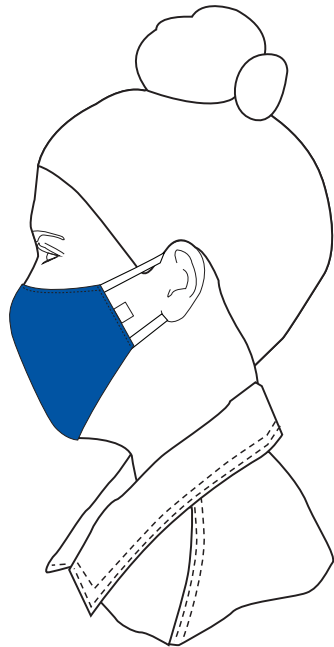

Converter strap may be purchased to convert ear loops to a head elastic

contact us at facemasks@goodwillindustries.ca

When you get your mask:

Remove from packaging.

Wash and dry mask prior to wearing as mask will shrink slightly.
Converter strap may be purchased for additional fit adjustments.

The best way to stop the spread of COVID-19 is to maintain social distancing with others outside of your household.



You may consider using a non-medical mask such as The Commuter to reduce the risk of transmission of COVID-19 in areas where physical distancing may be challenging.

Mask should not be placed on or used by: children under the age of two, anyone who has trouble breathing, anyone who is unconscious, incapacitated or otherwise unable to remove the mask without assistance.

Wash your hands before putting the mask on and after taking it off. Practise good hand hygiene while you are wearing the mask.

Mask should fit well around your nose and mouth. Avoid moving the mask around, touching the mask, or adjusting it often. Do not share it with others.

How to convert your mask from ear to head straps

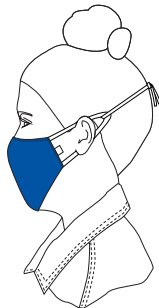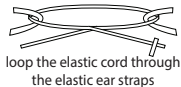

**Looking for a longer wearing mask with a filtration pocket?
Look for The Worker reusable face mask.**



reusable



washable



durable



safe

100% cotton/coton,
Machine wash / Lay flat to dry
Laver à la machine/Plat sec
Made in/Fait au Canada:
Goodwill Industries, Ontario Great Lakes
255 Horton Street, London, ON N6B 1L1